

DGHM/
VAH
listed

ÖGHMP
listed

Viruzidieliste
IHO
geprüft nach
Standardmethode
DIN EN

CE 0482

Disinfectant wipes

DESCOCEPT AF WIPES

alcoholic wipes for rapid disinfection

- aldehyde-free
- with fresh scent
- extensive effectiveness incl. Noro viruses

Product Properties

DESCOCEPT AF WIPES are ready-for-use, aldehyde-free disinfectant wipes based on ethanol and quaternary ammonium compounds. DESCOCEPT AF WIPES have a pleasant fresh scent. Discolourations of materials after use of DESCOCEPT AF WIPES are virtually impossible due to its aldehyde-free composition. The disinfectant solution is extensively effective against viruses incl. Noro virus. DESCOCEPT AF WIPES are the ideal surface disinfectant for areas which require a rapid wipe disinfection of alcohol resistant surfaces and inventory.

DESCOCEPT AF WIPES in the handy dispenser box can be extracted individually and are ready to use straight away.

Range of Application

Acc. to MDD (Medical Device Directive): Rapid disinfection of non invasive alcohol resistant medical devices.

Acc. to Biocidal Products Regulation: Rapid disinfection and cleaning of alcohol resistant medical inventory and all kinds of surfaces.

Application

Wipe surfaces and objects with **DESCOCEPT AF WIPES**. Surfaces must remain completely wet for the required contact time.

Application times

with reference to the soaking solution

Descosept AF:

Antimicrobial properties

The disinfectant solution **DESCOCEPT AF** is:

- bactericidal (incl. MRSA)
- tuberculocidal
- yeasticidal, fungicidal
- virus inactivating incl. HBV/HIV/HCV/BVDV/Rota-/Adeno-/Vaccinia-/Noro viruses



Antimicrobial properties		30 sec.	1 min.	2 min.	5 min.	30 min.
surface disinfection acc. DGHM/VAH (bactericidal, yeasticidal)	with mechanical action, dirty conditions		•			
	without mechanical action, clean conditions		•*			
	without mechanical action, dirty conditions			•*		
surface disinfection acc. DGHM/VAH (tuberculocidal)	without mechanical action, clean conditions			•		
active against all enveloped viruses incl. HBV/HIV/HCV	acc. RKI/DVV	•				
	BVDV	•				
	Vaccinia-Virus	•				
Adeno virus	acc. RKI/DVV		•			
Rota virus	acc. RKI/DVV		•			
Noro virus (MNV)	acc. EN 14476				•	
tuberculocidal (<i>M. terrae</i>)	acc. EN14348		•			
fungicidal (<i>Aspergillus brasiliensis</i>)	clean conditions					•



DESCOSEPT AF WIPES

alcoholic wipes for rapid disinfection

Contact times acc. to EN

Antimicrobial properties		30 sec.	1 min.
EN 1040	bactericidal	•	
EN 1275	yeastocidal	•	
EN 1276	bactericidal, clean conditions		•
EN 1650 (C. albicans)	yeastocidal, clean conditions		•
EN 13727	bactericidal, clean conditions	•	
EN 13624 (C. albicans)	yeastocidal, clean conditions		•
EN 13697	bactericidal, clean conditions		•
EN 13697 (C. albicans)	yeastocidal, clean conditions		•

Composition

100 g solution of DESCOSEPT AF contain as active ingredients:

42.0 g ethanol, 0.05 g didecylmethyl ammonium chloride

Contains perfumes.

Chemical/Physical Properties

(Data refers to the DESCOSEPT AF solution)

Appearance: clear, colourless liquid
pH-value: 7.0 +/- 1.0
Flash-point acc. to DIN 51 755: 25°C

Delivery Units

single unit	delivery unit	pharma. ID	REF-No.
box /120 wipes of 14 x 20 cm	12 x 1 box	1898486	00-312-D120
refill /120 wipes of 14 x 20 cm	12 x 1 bag	1898581	00-312-R120

Additional information

Flammable solid. If medical advice is needed, have product container or label at hand. Keep out of reach of children. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Keep container tightly closed. Do not breathe vapour. Use only outdoors or in a well-ventilated area. **IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Dispose of contents/container to approved disposal company or local collection.

Expert opinions

Expert reports refer to the disinfectant solution **DESCOSEPT AF**:

Dr. Pitten, Hygienist, Gießen: Expert's report on surface disinfection acc. to DGHM/VAH-guidelines.

Expert's report on tuberculocidal activity acc. to EN14348.

Dr. Suchomel, Hygienist, Vienna: Expert's report on surface disinfection acc. to DGHM/VAH-guidelines.

Dr. Steinmann, Virologist, Bremen: Expert's report on virucidal activity against Adeno virus, Vaccinia virus (BVDV), Rota virus.

Prof. Dr. Werner, Hygienist, Schwerin: Expert's report on the tuberculocidal activity. Expert's report on Noro virucidal (MNV) activity acc. to EN 14476. Expert's report on bactericidal and yeastocidal efficacy acc. to EN 1040, EN 1275, EN 1276, EN 13727, EN 13624, EN 1650 and EN 13697.

Dr. Brill, Hygienist, Hamburg: Expert's report on fungicidal activity (*Aspergillus niger*).

Dr. Voss, Dermatest, Münster: Expert's report on skin tolerance (epicutane test).

Konrad Hornschuch AG, Weißbach: Manufacturer's approval for use with Skai® cushion materials.

Bergé & Co. GmbH, Versmold: Manufacturer's approval for use Descosept AF with MM disinfection mats.

Product status/listings

The disinfectant solution **DESCOSEPT AF** is listed in the current disinfectant list of the VAH/DGHM and of the ÖGHMP.

The disinfectant solution **DESCOSEPT AF** is listed in the iho list of virucidal disinfectants (www.iho-viruzidie-liste.de)

DESCOSEPT AF WIPES is registered as a biocide product at BAuA (German Institute for Employment Protection and Occupational Medicine):

BAuA reg. No.: N - 39455.

DESCOSEPT AF WIPES conforms to the EU-guideline 93/42/EC for medical devices.

Environmental information

The products of Dr. Schumacher GmbH are manufactured according to modern, safe and environmentally friendly processes. Due to the compliance with high quality standards the constant and very good product quality can be guaranteed.

Use disinfectants safely. Always read the label and product information before use.