

DESCOSUC CLEANER

cleaner for dental suction systems

- ✓ free from aldehydes
- ✓ free from phenols



- excellent cleaning power
- low foaming
- additional intensive cleaner for use with *DESCOSUC*

PRODUCT PROPERTIES

Liquid concentrate for the regular intensive cleaning of dental suction systems with and without amalgam separators, dental sinks and lines. Desinfectant and cleaner for all kinds of dental suction systems.

DESCOSUC CLEANER guarantees an optimal cleaning and a trouble-free, smooth operation of the dental suction system.

APPLICATION

Apply **DESCOSUC CLEANER** once or twice a week. Depending on level of soiling use 50 ml to 100 ml **DESCOSUC CLEANER** and fill up to 1 litre with water. Prepare 2 litre of fresh working solution before cleaning procedure (cleaning can be done e.g. during lunch break). Extract 2/3 of the working solution through the suction tubes and pour 1/3 of the working solution into the dental sink.

The best cleaning result will be achieved at an exposure time of ca. 1.5 to 2 hrs. After exposure time rinse carefully with clear water in order to completely remove dissolved soiling from the tubes.

Do not leave the working solution in the suction system over night.

CONCENTRATIONS & EXPOSURE TIMES

normal soiling level: 5 %
high soiling level: up to 10 %

Exposure time: 1.5 to 2 hrs, depending on soiling level

CHEMICAL/PHYSICAL PROPERTIES

Appearance of the concentrate: clear, yellowish solution
pH-value (concentrate): ca. 1.7 +/-0.5
specific gravity (20°C): ca. 1.058 g/ml

CE

COMPOSITION

Organic acids, complexing agents, corrosion inhibitors, solubilisers.

DELIVERY UNITS

single unit	delivery unit	PZN	REF-No.
1 L bottle	10 x 1 L	-	00-818-010
5 L container	3 x 5 L	-	00-818-050

APPLICATION AIDS

Metering pump for container, dosage bottle, tap for container

SPECIAL INFORMATION

Causes severe burns. In case of contact with eyes rinse immediately with plenty of water and seek medical advice. This material and its container have to be disposed in a safe way. Wear suitable protective clothing, gloves and eye/face protection. Keep out of reach of children. Do not ingest. In case of accident or if you feel unwell, seek medical advice immediately (show the label where possible).

REGISTRATION / CERTIFICATION

Conforms to the EU-guideline 93/42/EEC for medical devices.

