

ECO WIPES

non-woven wipes dispenser system for surface disinfectants

- suitable for use with all surface disinfectants
- refillable dispenser system
- excellent release of the active substances
- very economical

The economical alternative!



CE

PRODUCT PROPERTIES

Dry wipes for impregnation with disinfectants and cleaners for rapid and easy disinfection and cleaning of surfaces and medical devices.

ECO WIPES are made of high quality lint-free non-woven material.

They can either be used in combination with ready to use rapid disinfectants or with a ready to use working solution of a disinfectant concentrate.

ECO WIPES are especially developed for surface disinfection.

They guarantee a perfect and even wetting of the surfaces.

ECO WIPES can be taken from a specially designed dispenser bucket **ECO WIPES VLIESTUCHSPENDER** by pulling them through a very practicable and re-closable dispensing system.

This system guarantees constant protection of the disinfectant solution from contamination and evaporation.

ECO WIPES DISPENSER SYSTEM

Dispenser bucket made of polyethylene and polypropylene, specially designed for the hygienic storage and easy removal of wet wipes. The wipes can conveniently be taken from a leak-proof and re-closable dispenser system specially developed for this application.

Dishwasher safe up to 60° C.

A special label to ease the documentation of the lot number, date of preparation and expiry date is supplied with each dispenser bucket.

APPLICATION

Prepare 2,5-3 litres (1 - 1,5 litres if using a 50-wipes roll) ready to use solution of the required disinfectant (concentrations and exposure times to be taken from manufacturer's instructions).

Then impregnate the wipes in the dispenser bucket with 2,5-3 litres (1 - 1,5 litres if using a 50-wipes roll) of the required disinfectant concentrate or with a ready to use alcoholic disinfectant.

It takes approx. 30 to 60 minutes until the wipes are completely saturated.

Please wipe surfaces and devices carefully and follow the exposure times of the disinfectant.

Besides all surface disinfectants of Dr. Schumacher, **ECO WIPES** can also be combined with products from other manufacturers. Please observe manufacturer's instructions for stability of surface disinfectants.

ECO WIPES in combination with the surface disinfectants of Dr. Schumacher GmbH can be used for up to 28 days after preparation of the dispenser system.*

ECO WIPES can be pulled off singly and should be taken from the opening system one after another. The following wipes are effectively protected by the covered collecting pan of the dispenser system.

Special labels which facilitate the documentation of the lot number as well as of the date of preparation and expiry date are supplied with each roll.

***Note:** The standing time of up to 28 days after preparation of the dispenser system is valid for all surface disinfectants of Dr. Schumacher GmbH as well as for the majority of similar products of which are not chemically stable in the working solution. In cases of any doubt about the chemical stability of a disinfectant solution, please contact us or the respective manufacturer of the product for further information.



ECO WIPES

non-woven wipes dispenser system for surface disinfectants

RANGE OF APPLICATION

ECO WIPES can be used for the cleaning and disinfection of surfaces in all areas of medical facilities such as hospitals, nursing homes and surgeries, in all areas of application in the pharmaceutical industry and laboratories as well as in the food industry and in canteens.

Especially applicable for:

- surfaces close to patients such as side tables, bedstands and stretchers
- all working surfaces
- areas where no mains water is available or a hygienic processing of cleaning implements is not possible

COMPOSITION


1 roll consists of 100 or 50 dry wipes respectively (32 cm x 30 cm) each made of high quality lint-free non-woven material. 100% syntetic fibre mixture.

DELIVERY UNITS

single unit	delivery unit	PZN	REF-No.
1 pack Eco Wipes roll with 100 wipes	3 x 1 roll	-	00-915-REW10003
1 pack Eco Wipes roll with 100 wipes	6 x 1 roll	-	00-915-REW10006
1 pack Eco Wipes roll with 50 wipes	3 x 1 roll		00-915-REW5003
Eco Wipes Dispenser System 1 dispenser bucket, unfilled	1 x 1 bucket	-	00-915-EEW001
Eco Wipes Dispenser System 6 dispenser buckets, unfilled	6 x 1 bucket	-	00-915-EEW002

APPLICATION AIDS

single unit	delivery unit	PZN	REF-No.
wall dispenser	1 x 1 bucket holder	-	00-902-EIM

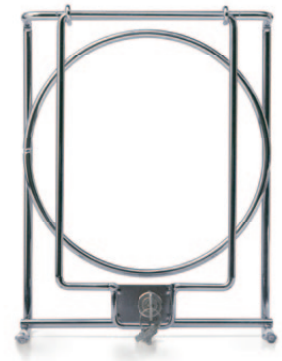

PRODUCT STATUS

Conforms to the EU-guideline 93/42 EEC for medical devices.

EXPERT'S OPINIONS

Dr. Wecker, Dr. Schumacher GmbH, Melsungen:
Suitability of the non-woven material of Eco Wipes for impregnation with surface disinfectants.

Dr. Brill, Hygienist, Hamburg:
Microbiological stability and standing time of **ECO WIPES** in combination with surface disinfectants Biguanid Fläche N, Cleanisept, Descosept AF, Descosept Pur, Optisept, Optisal N and Ultrasol F.

ENVIRONMENTAL INFORMATION

The products of Dr. Schumacher GmbH are manufactured according to modern, safe and environmentally friendly processes. Due to the compliance with high quality standards the constant and very good product quality can be guaranteed. The company is certified according to DIN EN ISO 13485, DIN EN 9001, DIN EN 14001 & BS OHSAS 18001 and has a validated eco management system (EMAS).

