



Dental

ORANGE SOLVENT

special cleaner for dental instruments

- made from natural orange oils
- with excellent cleaning power
- removes dental materials easily

Product Properties

ORANGE SOLVENT is a ready-for-use cleaning solution which thoroughly removes residues of plaster and pastes as well as plaster casts and cement from dental instruments and inventory. Even hard-resistant spots and stains such as oil spots or resin spots and stains on clothes or carpets can be removed with Orange Solvent. Very economical to use.

Range of Application

- removal of alginate- and cement residues as well as spots of zinc-eugenol
- for cleaning Unna's paste dressings and plaster casts
- residues of iodine and other colour signal spots
- undesirable stickers and old labels as well as adhesive residues can be removed easily after short exposure time
- removes oily spots from clothes, carpets and upholstery
- for removal of undesirable stripes and welts on PVC and linoleum floors
- removes residues of greasy pencils and permanent markers

Application

ORANGE SOLVENT on a cloth or cotton pad and rub into the areas to be treated.

Attention: The material compatibility can not be guaranteed for all synthetic materials - especially not for amorphous thermoplastics like acrylic glass or poly carbonate. The compatibility must be tested on non- visible areas before use.

Composition

Citrus Sinensis Peel Extract, Limonene, Decyl Oleate.

Delivery Units

single unit	delivery unit	pharma.-ID	REF-No.
250 ml bottle	9 x 250 ml	0792320	00-805-0025
500 ml bottle	6 x 500 ml	0792337	00-805-005

Product Status

Conforms to the EU guideline 93/42 EC for medical devices.

Special Information

Danger
Orange terpenes Flammable liquid and vapour. May be fatal if swallowed and enters



airways. Causes skin irritation. May cause an allergic skin reaction. Very toxic to aquatic life with long lasting effects. If medical advice is needed, have product container or label at hand. Keep out of reach of children. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Wear protective gloves/protective clothing/eye protection/face protection. IF SWALLOWED: Immediately call a POISON CENTER or doctor/physician. Do NOT induce vomiting. IF ON SKIN (or hair): Remove/Take off immediately all contaminated clothing. Rinse skin with water/shower. If skin irritation or rash occurs: Get medical advice/attention. Avoid release to the environment. Dispose of contents/container to in accordance with local and national regulations.

Environmental Information

Dr. Schumacher GmbH's products are manufactured according to modern, safe and environmentally friendly processes. Compliance with high quality standards guarantees a continuously excellent product quality. Schumacher GmbH is certified according to DIN EN ISO 13485, DIN EN ISO 9001, DIN EN ISO 14001 & BS OHSAS 18001 and has a validated eco management system (EMAS).

