

SPEZIAL ÖLSPRAY

✓ free from CFC's

maintenance spray for surgical instruments

- based on toxicologically harmless medical white oil
- for instrument maintenance prior to steam sterilisation
- optimal lubrication of moving instrument parts

CE



PRODUCT PROPERTIES

SPEZIAL ÖLSPRAY is an aerosol spray for the maintenance of medical instruments, especially for articular instruments. After the treatment with **SPEZIAL ÖLSPRAY**, medical instruments can be subjected to the steam sterilisation. For the maintenance of hinges and valves of rigid endoscopes. The special spray valve provides a fine, even protective layer. A subsequent sterilisation after the application on the instruments is not affected by this protective layer. **SPEZIAL ÖLSPRAY** preserves the value of your instruments because of an optimal lubrication and an effective corrosion inhibition. Due to the use of paraffinum liquidum, **SPEZIAL ÖLSPRAY** is compatible with most materials and toxicologically harmless according to drug and food law.

RANGE OF APPLICATION

For the maintenance of

- medical instruments (e.g. prior to steam sterilisation)
- turbines
- hinges, handpieces and angle pieces
- non-maintenance-free valves of rigid endoscopes

SPEZIAL ÖLSPRAY guarantees an optimal lubrication of moving parts of instruments etc. Due to the formation of an even protective layer on the surface of steel devices, an effective protection against corrosion is guaranteed.

APPLICATION

Instruments:

Spray the half-opened instruments with **SPEZIAL ÖLSPRAY** from a distance of about 25 cm. If indicated sterilise the instruments afterwards.

Attention: Do not apply too much oil, in order not to interfere with sterilisation. Only a thin film should remain.

Hinges, turbines or handpieces and angle pieces:

Use **SPEZIAL ÖLSPRAY** two times a day or spray it through the devices for app. 1 sec before sterilization or autoclavation respectively. To apply the product hold the spray bottle vertically. Shake before use.

COMPOSITION

Paraffinum liquidum

Propellants: propane/butane (free from CFC's)

DELIVERY UNITS

single unit	delivery unit	PZN	REF-No.
500 ml aerosol can	12 x 500 ml	-	00-108-005

SPECIAL INFORMATION

Container is under pressure. Avoid sunray and temperatures exceeding 50°C. Do not open the bottle even after use and do not incinerate. Highly inflammable.

Do not spray into flames or onto hot surfaces. Keep away from ignition sources - no smoking.

In case of insufficient ventilation the formation of explosive mixtures is possible.

If **SPEZIAL ÖLSPRAY** is used on latex or rubber parts, these materials may swell; in these cases use **SILIKON SPRAY**.

Do not use **SPEZIAL ÖLSPRAY** as lubricant for flexible endoscopes.

PRODUCT STATUS

Conforms to the EU-guideline 93/42 EEC for medical devices.



DR. SCHUMACHER GMBH
Am Roggenfeld 3, D-34323 Malsfeld
Telephone +49 5664 9496-0

WWW.SCHUMACHER-ONLINE.COM